



ThinkSAFE Maldives Pvt. Ltd.

Philips HeartStart OnSite AED

Philips HeartStart Defibrillators enable virtually anyone to treat the most common cause of SCA by delivering a shock quickly and effectively, wherever it happens—at work, at play, while traveling—providing the power to save a life.



Ready Out Of The Box

The Philips HeartStart OnSite Ready-Pack configuration is virtually ready to rescue out of the box. Enjoy peace of mind knowing your device is deployed correctly and is ready when needed:

- Arrives with pads cartridge and battery already installed.
- Just pull the green tab to launch the

initial self-test.

- Automatic daily self-tests, including pads, help ensure continued readiness

How To Use

Using the HeartStart OnSite Defibrillator is simple. Pulling the green handle activates the defibrillator and its voice instructions and visual icons. These instructions are paced to your actions, to help guide you through the entire process—from placing each pad on the patient to delivering a defibrillation shock and performing CPR.

The cartridge contains two adhesive pads that are placed on the patient's bare skin as indicated by the pictures on the pads. The pads are "smart" because they sense when they have been removed from the cartridge and when each has been applied to the patient, adjusting the voice instructions to your actions.

The HeartStart OnSite can be used on patients of any age, including infants and children. OnSite senses when the special infant/child SMART Pads Cartridge is installed. It automatically adjusts to a lower energy level more appropriate for infants and children, and also provides coaching for performing infant/child CPR.

To practice your skills, a special training pads cartridge (adult or infant/child) can be installed in the defibrillator. It disables the defibrillator's ability to shock, while walking you through patient care scenarios.

Documents

- Philips Heartstart OnSite Brochure
- Philips Heartstart OnSite Technical Reference Manual
- Philips Heartstart OnSite Owners Manual (English)
- Philips Heartstart OnSite Owners Manual (Spanish)
- Philips Heartstart OnSite Data Sheet
- Philips Heartstart OnSite Accessories Catalog
- Philips Heartstart OnSite Warranty Statement

Package Contents

- Philips Heartstart OnSite AED
- Battery Pack
- SMART Pads cartridge
- Quick set up guide
- Quick reference guide

- Owner's manual
- Optional carry case not included, but available.
- Free! AED CPR Responder Kit
- Free! AED Decal Sticker
- Free! AED Inspection Tag
- Free! Ground Shipping

Technical Specifications

Weight	3.3 lbs	Energy Protocol	150-150-150
Dimensions	7.4" x 8.3" x 2.8"	IP Rating	IP21
Prompt Type	Voice	Warranty	8 years
Operation Mode	Semi-Automatic	ECG Display	No

* Warranties mentioned are only for reference purposes. ThinkSAFE Maldives does not provide any warranty.